

ivNowtm

Patent Pending



Modular pod configuration

Wall-mount: 1-6 pods

Counter-top: 1-3 pods

Pole mount: 1 pod

Anodized aluminum heat plate

Lightweight, medical grade plastic housing

Intelligent bag sensors

Dual temperature sensors

Heats to 40°C (104°F)

Accuracy of +0°/-1.7°C (+0°/-3°F)

 **ENTHERMICS**
Medical Systems

www.enthermics.com



*Experience the Entermics difference:
Quality beyond comfort.*

ivNow™ Modular Fluid Warming Pod

The ivNow fluid warmer quickly warms and maintains the temperature of injection/IV and irrigation solutions prior to their use. The specially contoured warming module cradles solution bags up to 1 liter in size. The unit is controlled by one (1) power switch and individual electronic controls with L.E.D. display for each pod. The control can easily be set to display temperatures in Celsius or Fahrenheit. A SmartSensor in the heating plate detects the presence of a bag and engages the heating mechanism to quickly begin warming the fluid. Two (2) temperature sensors work in unison to precisely and continuously read the temperature of the bag and another sensor monitors the plate temperature. A green ready light will illuminate when fluid is within $\pm 0/-1.7^{\circ}\text{C}$ ($\pm 0/-3^{\circ}\text{F}$) of the set point temperature, 40°C (104°F). The heater will reengage as necessary to maintain the temperature within $\pm 0/-1.7^{\circ}\text{C}$ ($\pm 0/-3^{\circ}\text{F}$) of the set point. The electronic control monitors the length of time the bag has been held at temperature, beginning when the bag reaches set point temperature. A recall button will display the time the fluid has been held at temperature.

ivNow-1

Fluid Warming in 70°F (21°C) Ambient Temperature (104°F set-point)

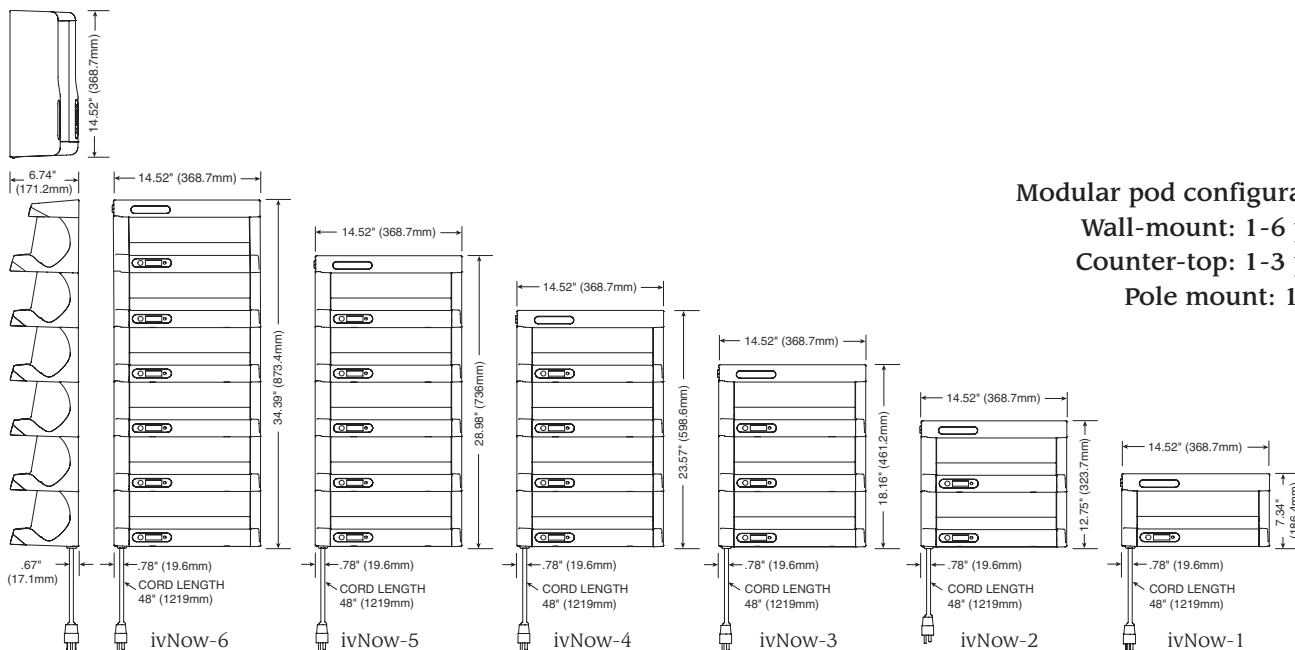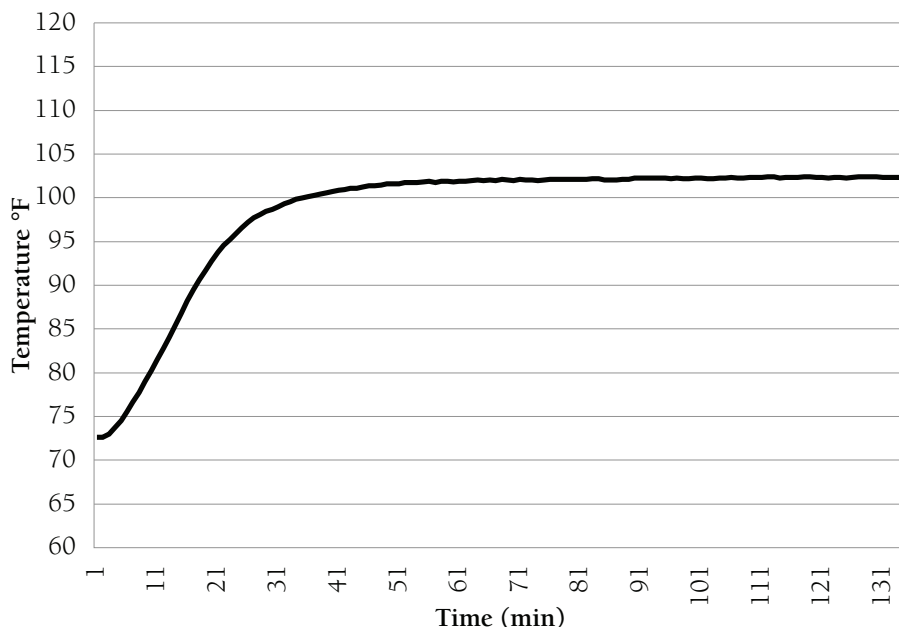

	ivNow-6	ivNow-5	ivNow-4	ivNow-3	ivNow-2	ivNow-1
Height	34.39"	28.98"	23.57"	18.16"	12.75"	7.34"
Width	14.52"	14.52"	14.52"	14.52"	14.52"	14.52"
Depth	6.74"	6.74"	6.74"	6.74"	6.74"	6.74"
Electrical						
Volts	120	120	120	120	120	120
Amps	7.50	6.25	5.0	3.75	2.50	1.25
Watts	900	750	600	450	300	150



ENTHERMICS
Medical Systems
www.enthermics.com

W164 N9221 Water Street • PO Box 443
 Menomonee Falls, WI 53052-0443
 Phone 262.251.8356 • 800.862.9276
 Fax 262.251.7067
 Email generalinfo@enthermics.com