# PROTEC®

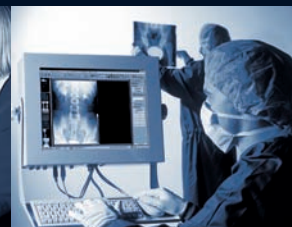
medical systems

## PROSCAN 35 CR-System

Speicherfolien-System  
CR-System

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*Easy changeover for  
the radiographer because  
the handling is comparable  
to that of films*

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*PROSCAN is a modern, high resolution scanner for imaging plates. It transfers x-ray information saved on the imaging plate into a digital signal. The CONAXX Software (included in the standard delivery) operates the CR-Scanner and routes the images automatically to a PACS. In case you do not have a PACS yet, our PROMIS medical imaging software would be a favourable alternative.*

#### **Rapid imaging for superior quality**

*Due to the fast scanning process, x-ray images are provided quickly for a reliable diagnosis. Misdosage and as a result overexposure or underexposure are almost impossible with this CR-technology due to the high exposure tolerance.*

#### **Small investment**

*The worthwhile investment brings the immediate benefit of no longer needing to work with chemicals, the disposal of chemistry and the cleaning of a processor. Expensive printouts on film are not required, diagnosis can be done fast and reliably on a diagnostic monitor. The imaging plates can be re-used and you can still continue to use your existing x-ray unit.*

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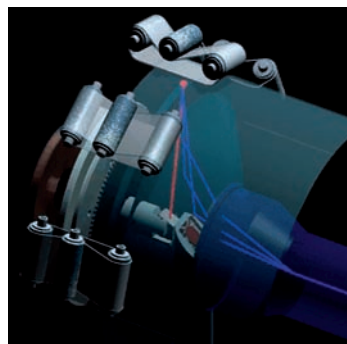
*Reliable diagnosis  
on the monitor (even  
with remote access).  
Printouts are not needed*

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*Highest precision due  
to state of the art  
technology and certified  
quality management*

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*Invest now and benefit from the  
advantages of CR-technology*

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**New: PROSCAN 35  
with integrated eraser for IPs**

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### Human medicine

*In human medicine the PROSCAN CR-System could be used either as independent modality, integrated in a PACS environment via CONAXX or as a stand alone solution for a private practitioner.*

*Through the operation of the CR-Scanner and image distribution with CONAXX Software the PROSCAN can be used very flexibly. The images are just routed to an existing PACS. If there is no PACS available yet, a system for diagnosis and archiving of images can be customized according to the needs of the practice or clinic with our modular structured PROMIS medical imaging software. Connections to patient/practice administration software can easily be realized, too.*

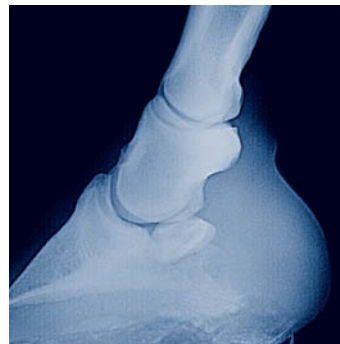
### Veterinary medicine

*Veterinarians appreciate the advantages of the PROSCAN CR-System as well. Especially when it comes to mobile x-rays. The uncomplicated handling and the high mobility (transportation case is available as an accessory) simplify the workflow when x-raying large animals. A diagnosis can be made on site immediately after taking the x-ray (e.g. on a laptop monitor) and further treatment can be initiated right away.*

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*Best image quality for all standard x-ray applications in human medicine*

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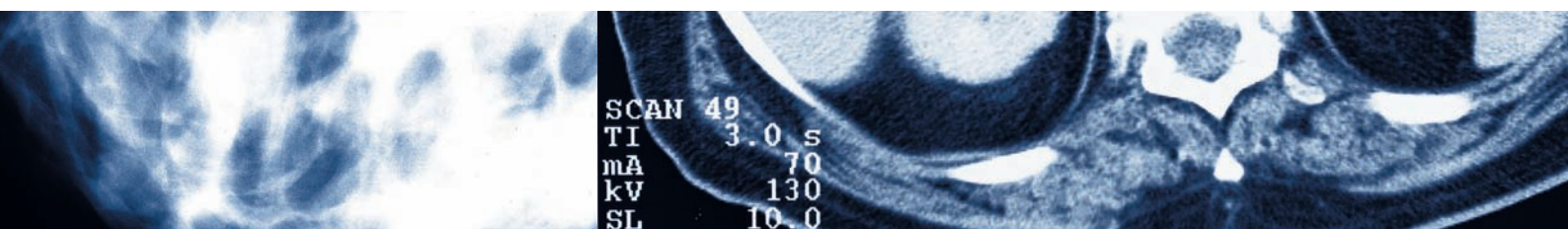
*PROSCAN offers affordable CR-Technology for the ambitious veterinarian*

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*Due to its compact design the PROSCAN is suitable for mobile applications*

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Simple workflow with  
PROSCAN and CONAXX

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**PROSCAN – patented technology of tomorrow!**

*The revolutionary PCS-technology (Photon-Collecting-System) ensures extraordinary low image noise by scanning the imaging plate with almost 100% efficiency of active radiation. Due to the very high resolution of more than 5 line pairs per millimetre an exact diagnosis is ensured. An outstanding feature is the greyscale resolution of 16-Bit.*

*The PROSCAN CR-System uses a centric laser beam, which is guided towards the image plate by means of a rotating pentagonal prism. The high rotation speed of this prism in connection with the centrally located digital sensor (PMT) that collects the luminescence radiation (emitted by the IP after being activated with the laser) and the linear drive that gently feeds the IP through the scanner ensure the outstanding resolution. Immediately after scanning the image data is automatically erased from the imaging plate. The IP can be used for the next x-ray right away, erasing as an additional step is not required.*

**Standard-Einstellungen / Standard settings:**

Format / Format	Auflösung / Resolution	Pixel matrix
18 x 24 cm	10 pixel/mm	1800 x 2400 Pixels
24 x 30 cm	10 pixel/mm	2400 x 3000 Pixels
35 x 35 cm	10 pixel/mm	3500 x 3500 Pixels
35 x 43 cm	10 pixel/mm	3500 x 4300 Pixels

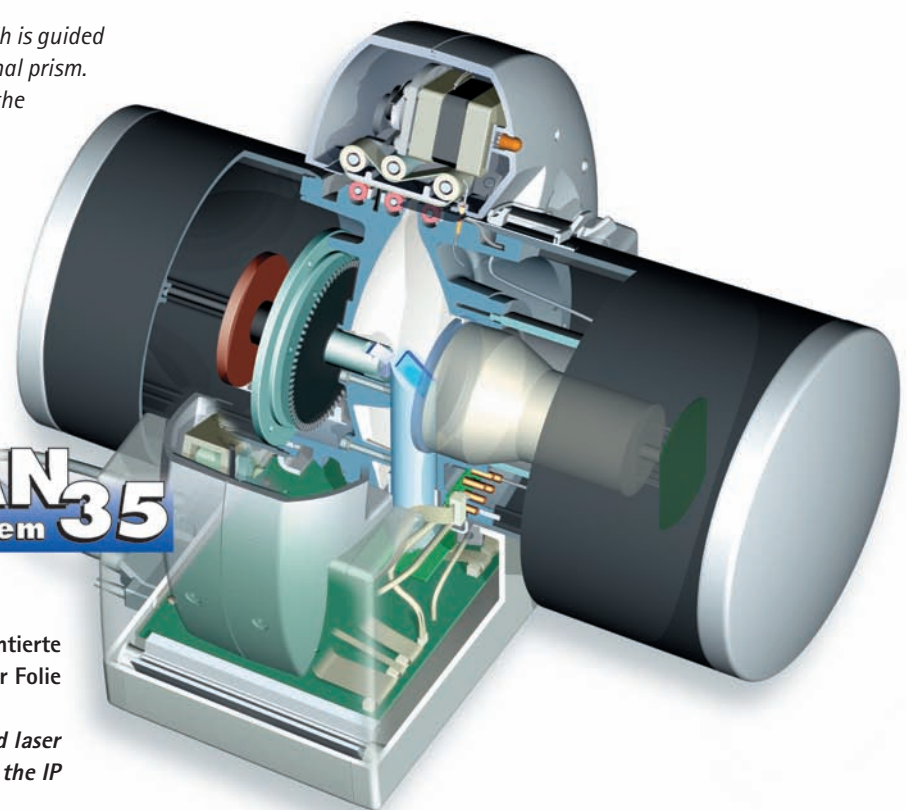
**High Resolution-Einstellungen / High resolution settings:**

Format / Format	Auflösung / Resolution	Pixel matrix
18 x 24 cm	15 pixel/mm	2700 x 3600 Pixels
24 x 30 cm	15 pixel/mm	3600 x 4500 Pixels
35 x 35 cm	15 pixel/mm	5250 x 5250 Pixels
35 x 43 cm	15 pixel/mm	5250 x 6450 Pixels

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*Scanning in a matter of seconds with patented laser technology and immediate erasing of image data from the IP*

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*The virtual waiting room  
provides an overview of  
all upcoming x-ray jobs*

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*Body part selection  
supports creation of  
individual scan jobs*

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*The patient's job  
management screen  
provides a status-overview  
of the individual jobs*

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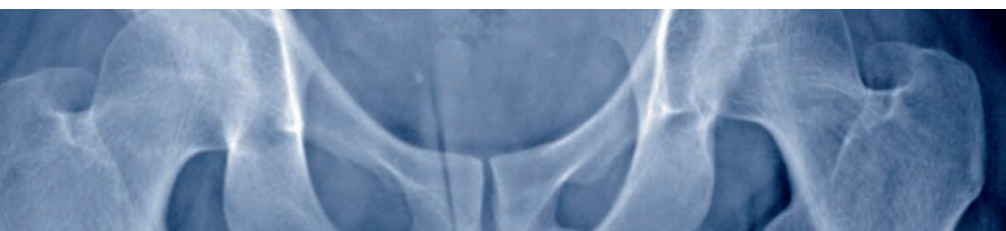
*CONAXX – the user friendly  
connection between PROSCAN  
and PACS*

### CONAXX Acquisition System

The CONAXX Acquisition System is included in the standard delivery and operates the complete PROSCAN CR-System. On a touchscreen (optional accessory) the software guides you in a user-friendly way through the complete workflow according to the process in a practice or clinic. The CONAXX Acquisition System consists of the Image Acquisition Module as well as DICOM Store, GDT Data Input/Response and E-mail Modules. With CONAXX multi-clinic applications, e.g. when doctors share modalities, can easily be realized (optional Multi-Clinic-Module). In addition it offers the following functions:

#### Patient acceptance / identification

- manual entering of new patients
- acceptance from patient/practice administration systems via GDT
- automatic worklist acceptance (SCU) including transfer of jobs (DICOM Input Module, optional)
- DICOM Query (part of optional DICOM Input Module)

#### Creation of x-ray jobs

- creation of individual scan-jobs with body part selection, incl. allocation of correct scan mode
- automated creation of multiple scan-jobs in one step
- general and patient orientated job management

#### Image preview and optimization

- realtime image preview
- "crop"-function (cutting of images for data reduction)
- comments, turn, flip, mirror, L/R-signs and annotations
- automatic window levelling and visual histogram
- brightness and contrast
- selection of several standard filters
- x-ray journal with automatic transfer of dose-area-product (optional)

#### Data transfer and archiving

- creation of DICOM-file with header information
- feedback to worklist-server (DICOM Input Module) or patient/practice administration system (GDT Data Input/Response Module incl.)
- image export as DICOM or in other formats (JPG, TIFF, BMP)
- DICOM Store - transfer of images to a DICOM compliant PACS (e.g. PROMIS)
- internal "safety-archive" with automatic memory adjustment
- printing images on Windows printer or sending images by email

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*In the General Job Management jobs can be selected according to their status*

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*In the preview the image can be processed and optimized*

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*At the end of the workflow further steps are defined (e.g. routing to PROMIS)*

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### Technical Data

Resolution:	varies depending on size and type of imaging plate
Smallest physical pixel size:	12,5 µm (respectively 40 Lp/mm)
Max. infeed width:	35 cm (14")
Dimensions (H/W/D):	39 cm x 38 cm x 52 cm (15 1/2" x 15" x 20 1/2")
Weight:	20,4 kg
Installation:	table top
Noise:	approx. 49 dB(A)
Temperature range:	+10 up to +30 °C (+50 up to +86 °F)
Power supply:	100-240 V / 50-60 Hz
Electrical connected load:	120 W
Laser protection class:	I (EN60825-1)
Further detail:	integrated printer port

### Interfaces:

- Integration in existing PACS / RIS / HIS
- Takeover of worklists (DICOM)
- Connection to existing system via GDT/BDT

### Standard delivery includes:

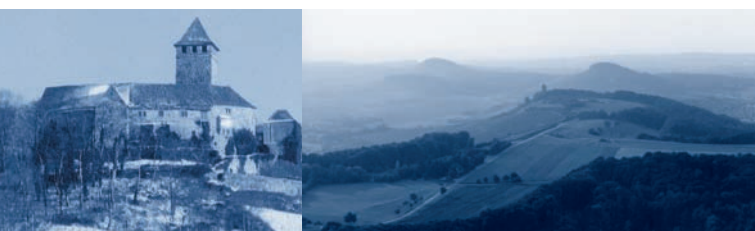
- CONAXX software, including the following modules:  
Standard, DICOM Store, GDT Data Input/Response & E-mail
- 2 imaging plates 18 x 24 cm / 24 x 30 cm, at buyer's option (8" x 10" / 10" x 12")
- 2 imaging plates 35 x 35 cm / 35 x 43 cm, at buyer's option (14" x 14" / 14" x 17")
- 8 protection sleeves (according to chosen formats)
- Connecting cable (3m) for USB and power supply

### Additional requirements (accessories):

- PC
- Diagnostic monitor
- Imaging storage
- Film cassettes without intensifying screens

### Recommended PC-system requirements:

- 2.0 GHz processor
- 1 GB main memory (RAM), better 2 GB
- 80 GB free hard disk space (recommendation >400 GB)
- Operating system Windows 2000™, Windows XP™ or Windows Vista™
- USB 2.0
- optional: TCP/IP-connection to network

PROTEC GmbH & Co. KG · medical systems  
Lichtenberger Straße 35 · D-71720 Oberstenfeld  
Tel.: +49 (0) 70 62/92 55-0  
Fax: +49 (0) 70 62/2 26 85  
e-mail: [protec@protec-med.com](mailto:protec@protec-med.com)  
<http://www.protec-med.com>

Hier entsteht innovative  
Medizintechnik.  
*This is where innovative  
medical technology is produced.*